

Thank You!

Thank you for purchasing the Peak Taima Rollator with Gutter Arms.

The Taima Gutter Arm Rollator is exceptionally light, strong and can easily be folded and stowed away. It provides ideal support during all your daily ventures as well as a seat to rest.

Before using the rollator for the first time please read this manual carefully and be sure to familiarize yourself with all the features of this product.

The user should keep this manual available at all times and keep it with the product if it is passed on to a third party. The manufacturer reserves the right to make changes to the information provided in these instructions as a result of technical developments and improvements.

This document may not be reprinted, translated or duplicated in any form whatsoever, in part or in full, without the manufacturer's prior written consent. There is no update service for this user manual. Please contact the manufacturer to keep abreast of new developments.

Peak Care Equipment
149 Princes Highway
Unanderra NSW 2526
Tel: +612 4272 2688
www.peak-care.com.au

Distributor:

Peak Taima Rollator with Gutter Arms

Product Codes: C4507-B-FX, C4505-E-FX



User Manual

1. Precaution

- The TAIiMA Gutter Arm Rollator is intended as a walking aid for persons with impaired mobility who weigh up to 150 Kg. It also provides a limited seating function for temporary rest.
- The rollator should be used only on firm, flat surfaces. It is not intended to be used for any other purpose - in particular as a climbing aid or to carry heavy loads. Any such use can be dangerous!

2. Safety Information

- Do not hang bags on the handles as this could cause the rollator to tip over.
- Use the service brake (by pulling the brake lever out) on inclines.
- Only use the rollator on firm, flat surfaces.
- Do not lean too heavily on the handles. Otherwise, the Rollator could tip over backwards.
- Be careful not to trap your fingers or clothes when folding up the rollator.
- Upholstered parts become hot when exposed to direct sunlight and can cause damage to the skin if you touch them. These parts should be covered over
- Alternatively, keep your rollator away from direct sunlight.

3. Warranty

- The TAIiMA Gutter Arm Rollator a 12-month warranty. Parts subject to wear are excluded from warranty. Warranty does not cover any sort of misuse of your rollator

Product Code	Overall Max. Height	Overall Min. Height	Overall Length	Overall Width	Seat Depth	Seat Width	Seat Height	Folded Width	Castor Size	Product Weight	Product Capacity
C4507-B (Small)	1226mm	1126mm	630mm	580mm	220mm	390mm	520mm	220mm	194mm	9Kg	150Kg
C4505-E (Tall)	1296mm	1138mm	670mm	630mm	220mm	440mm	592mm	220mm	194mm	9Kg	150Kg

4. Setting Up & Folding The Rollator

Place the rollator in front of you in the direction of travel. Using both hands, press the tubes on either side of the seat down to the left and right until the seat clicks into place.
Pull up the seat by its folding strap and the Rollator will easily fold up.

5. Parking Brake

Press the brake lever forward for Parking and pull it back to release it.

6. Adjustment of Armrest & Height

Adjustment of the Armrest height and depth can be adjusted by using the Levers shown in picture can be achieved.

7. Cleaning:

Clean and disinfect the rollator at regular intervals. To do so, use mild soapy water with no scouring additives. You can disinfect the rollator using commercially available products. Never use steam degreasers. Never use scouring or caustic cleaning agents.

8. Maintenance:

The TAIiMAGutter Arm Rollator requires no maintenance. If the braking function weakens, wheel movement becomes stiff or you notice any other functional impairment, please take the Rollator to your dealer for service.

9. Reuse by third parties:

If you pass on your Rollator to a new user, please remember to hand over all the technical documentation needed to ensure safe operation. In accordance with specifications, the rollator must first be inspected, cleaned and returned in pristine condition by a specialist dealer.